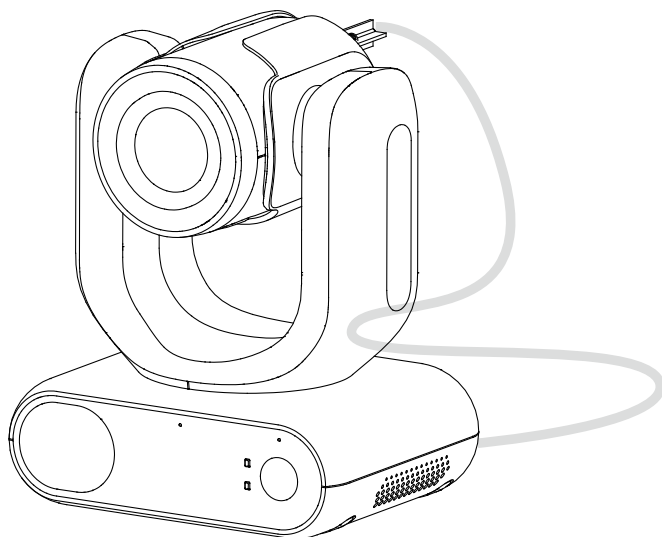
AVer

MD330U Series PTZ Camera

— **User Manual** —

How to Clean and Disinfect

| | |
|--|---|
| Warning Risk of personal injury | Do not use cleaning agents containing phenol, as phenol may cause skin burns if not rinsed thoroughly and does not contain sufficient cleaning or disinfectant properties. |
| Caution | <ol style="list-style-type: none">1. The camera is designed for easy cleaning and disinfection.2. Before cleaning the camera, disconnect it from the power source.3. Cleaning procedures should only be carried out by personnel familiar with camera operation.4. Do not spray cleaning fluid, especially water, into any internal electrical equipment or parts of the camera to avoid possible short circuits, corrosion, malfunction and electrical shock hazards to users or service personnel.5. Corrosive cleaning agents may cause discoloration or damage the camera. Before using any cleaning agent, test it in an inconspicuous area. |
| When | Clean the camera before and after use. |
| Steps | <ol style="list-style-type: none">1. Disconnect the camera from a power source.2. Cleaning personnel must wear cleaning gloves.3. Before using cleaning alcohol, make sure it has not expired.4. Use a cleaning cloth dampened with cleaning alcohol (75%).5. Wipe any surface that may come into contact with the camera.6. After wiping the camera, avoid contact with bare hands. |
| Frequency | No regular cleaning is required. Clean before and after use. |

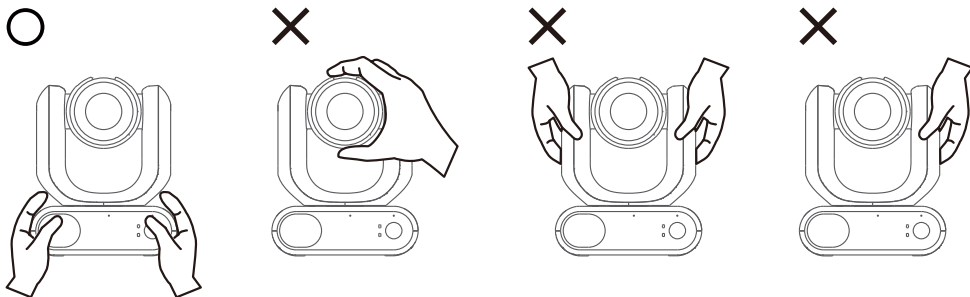
Precautions for Use

AVer does not accept any responsibility for accident or damage during installation if users do not follow the procedures described in this manual.

The AVer MD330U(I) Series PTZ Camera is a high definition medical grade camera designed for physicians and caregivers, and suitable for use in a hospital examination room, patient monitoring and similar medical environments. This non-contact device can be operated by physicians and caregivers without physical contact with patients, to document and update patients' injuries by taking snapshots or record videos.


To installation and operation personnel

The required information including the device's hardware parts, operations and important notice has been written and illustrated in this manual. Except for the information in this manual, no special skills, training and knowledge are required for the operators. Read the user manual thoroughly and operate the device correctly and safely. For common issues, please refer to the <Troubleshooting> chapter. For further assistance, please contact AVer technical support.

• Transportation

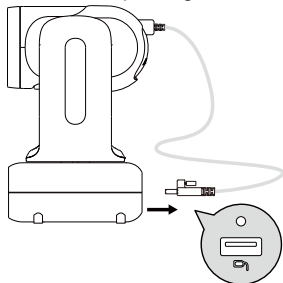
- Hold the bottom of the device with both hands. Do not grab the lens or the lens holder.
- When transporting, unplug the lens cable lens from the lens cable port on the bottom of the device and package the device in its original box.

- **Operating conditions**

This device is for indoor use only. Do not install or use the device in the following conditions to avoid malfunction or damage.

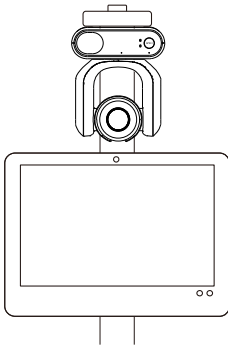- Do not exposed to direct sunlight.
- Do not exposed to rain or water.
- Do not exposed to extreme temperatures. Recommended operating temperatures: 0°C to +35°C; humidity: 20% to 80%.
- Do not install the device sideways on a wall.
- At sea, in coastal areas or in locations where corrosive gases are emitted.
- Locations subject to strong vibrations, such as in vehicles or on carts travelling on uneven surfaces. The lens may fall off under strong vibrations.


- **To ensure the device perform stably over a prolonged period**


- Do not use this unit in a location with high temperature and humidity levels, which may cause its parts to deteriorate and shorten its service life.
- Do not subject the device to sudden changes in temperature. Keep a cooling unit or heating unit away from this unit.
- Do not leave the device with the camera facing the sun.

- **Operation instructions**

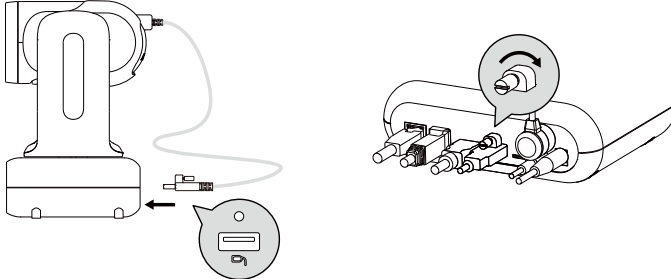- Connect the lens cable to the lens cable port on the bottom of the device and tighten the lens cable screw to secure.

- Do not connect the lens cable to computers, USB ports, or USB extension cables as doing so will damage the device. The lens cable is not a standard USB cable.
- Make sure the lens cable installed behind the lens is untangled before powering on the device.
- Do not detach the lens in non-Handheld Mode. Please refer to the <Mode Switch> chapter.
- When installing the lens in Handheld Mode, make sure the lens is securely inserted into the lens holder and the latch is properly locked. Please refer to the <Mode Switch> chapter.
- Use the device with care. Do not operate the device beyond its designed limits to avoid damage.

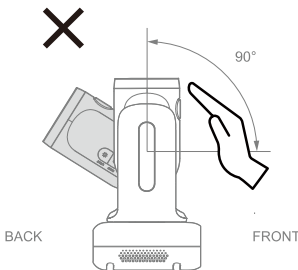- **Always use the supplied power adapter and power cord to connect the device to a power outlet**

- Before installation, make sure to disconnect the device from a power outlet.
- Install the device near the power outlet to make sure the power plug can be plugged and unplugged from the power outlet easily.
- To avoid electric arc from damaging other electrical components, connect the power cord to the DC 12V power jack on the device before connecting to a power outlet.
- The length of the power cord should be less than 3 m.
- Do not place the device where the cord can be stepped on as this may fray or damage the cord or the plug.

- **Powering off the device**

- The device does not have a power switch. Disconnect the device from a power outlet before maintenance, cleaning or in the event of an emergency.

- **Maintenance**

- The device does not have a power switch. Disconnect the device from a power outlet before maintenance.
- Designate personnel to clean the device. Please refer to the <How to Clean and Disinfect> chapter.
- When the device surface is stained, please refer to the <How to Clean and Disinfect> chapter.
- Regular maintenance keeps your device operational and safe. Check all parts, including the detachable lens, camera unit and remote control.

- **Disposal of the device**

- When the device is no longer used, turn off the device and dispose of it properly.
- When disposing of the device or accessories, follow relevant regulations in your country or region and medical facility regarding environmental pollution.

Symbols on this Product

The symbols on this product, including the accessories, represent the following.

| | |
|---|---|
|  | <p>The WEEE symbol. This symbol indicates that this product must not be disposed of with your other household waste. Instead, you need to dispose of the waste equipment by handing it over to a designated collection point for the recycling of waste electrical and electronic equipment. For more information about where to drop off your waste equipment for recycling, please contact your household waste disposal service or the shop where you purchased the product.</p> |
|  | <p>The CE compliance logo. This logo indicates that the product conforms to the relevant guidelines/standards for the European Union harmonization legislation.</p> |
|  | <p>The FCC compliance logo. This logo indicates that the product conforms to Federal Communications Commission compliance standards.</p> |
|  | <p>The UKCA (UK Conformity Assessed) symbol. This symbol indicates that a product placed on the Great Britain market meets the UKCA Marking requirements.</p> |
|  | <p>The RCM compliance logo. This logo indicates that the product conforms with Australian RCM guidelines.</p> |
|  | <p>This logo is intended to alert the users to the presence of un-insulated “dangerous voltage” within the product’s enclosure that may be of sufficient magnitude to constitute a risk of electric shock to persons.</p> |
|  | <p>This logo is intended to alert the users to the presence of important operating and maintenance (servicing) instructions in the literature accompanying the appliance.</p> |
|  | <p>The China RoHS symbol. The figure of this symbol represents the number of years during which no hazardous substances would leak or mutate under normal conditions of use.</p> |
|  | <p>The ISO 7010-M002 symbol. Refer to instruction manual/booklet.</p> |
|  | <p>The alternating current symbol. This symbol indicates that the power input/output for the product is alternating current.</p> |
|  | <p>The direct current symbol. This symbol indicates that the power input/output for the product is direct current.</p> |

Warning

To avoid the risk of electric shock, this equipment must only be connected to a supply mains with protective earthing. Do not modify this equipment without authorization of the manufacturer.

Caution

Possibly hazardous optical radiation emitted from this product. Do not stare at operation lamp. May be harmful to the eyes.

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Note: Product version (MED SKU or IT SKU) may vary by region. Please consult your local sales representative or technical support for specific details.

● MED SKU

Federal Communication Commission Interference Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

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Federal Communications Commission

NOTE: This equipment has been tested and found to comply with the limits for a Class A digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference when the equipment is operated in a commercial environment. This equipment generates, uses, and can radiate radiofrequency energy and, if not installed and used in accordance with the instruction manual, may cause harmful interference to radio communications. Operation of this equipment in a residential area is likely to cause harmful interference in which case the user will be required to correct the interference at his own expense.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following two conditions:

- (1) This device may not cause harmful interference, and
- (2) this device must accept any interference received, including interference that may cause undesired operation.

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Warning

This is a class A product. In a domestic environment this product may cause radio interference in which case the user may be required to take adequate measures.

Caution

Risk of explosion if battery is replaced by an incorrect type. Dispose of used batteries according to the instructions.

PSTI Statement of Compliance

Please refer to the following website: <https://www.aver.com/product-security-advisory>

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DISCLAIMER

No warranty or representation, either expressed or implied, is made with respect to the contents of this documentation, its quality, performance, merchantability, or fitness for a particular purpose. Information presented in this documentation has been carefully checked for reliability; however, no responsibility is assumed for inaccuracies. The information contained in this documentation is subject to change without notice.

In no event will AVer Information Inc. be liable for direct, indirect, special, incidental, or consequential damages arising out of the use or inability to use this product or documentation, even if advised of the possibility of such damages.

TRADEMARKS

“AVer” is a trademark owned by AVer Information Inc. Other trademarks used herein for description purpose only belong to each of their companies.

COPYRIGHT

©2026 AVer Information Inc. All rights reserved. | May 14, 2026

All rights of this object belong to AVer Information Inc. Reproduced or transmitted in any form or by any means without the prior written permission of AVer Information Inc. is prohibited. All information or specifications are subject to change without prior notice.

All HDMI trademarks and trade dress are registered trademarks or trademarks of HDMI Licensing Administrator, Inc.

More Help

For FAQs, technical support, software and user manual download, please visit:

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Contents

| | |
|--|-----------|
| How to Clean and Disinfect | 2 |
| Precautions for Use..... | 3 |
| Overview | 1 |
| Package Contents | 1 |
| Optional Accessories | 1 |
| Parts Info | 2 |
| LED Indicators | 3 |
| Dimensions..... | 4 |
| Pan and Tilt Angle..... | 4 |
| Change Your Lens Cable..... | 5 |
| Switch between Standard and Standard Short Cable | 5 |
| Switch from or to a Long Cable..... | 6 |
| Detach or Install the Lens..... | 7 |
| Remote Control..... | 8 |
| Shortcuts..... | 9 |
| Connections..... | 10 |
| RS-232 Pin Definition | 11 |
| Set Up Your Camera | 12 |
| OSD Menu | 12 |
| IP Address Setup..... | 12 |
| Static IP | 12 |
| DHCP..... | 13 |
| OSD Menu Tree..... | 14 |
| Access the Web Interface..... | 17 |
| AVer Device Utility..... | 17 |
| AVer ViewCare..... | 19 |
| Web Interface | 20 |


| | |
|--|-----------|
| Live View | 20 |
| Camera Control..... | 20 |
| Preset | 21 |
| AI Setting | 22 |
| Eyes Tracking | 22 |
| AI Video Detection | 23 |
| Camera Settings..... | 25 |
| Exposure..... | 25 |
| Image Process | 26 |
| Video & Audio | 27 |
| Beamforming Angles..... | 28 |
| Noise Suppression Performance | 29 |
| Network | 30 |
| System..... | 32 |
| Room Management | 35 |
| Configure Settings on Room Management..... | 35 |
| Specifications | 36 |
| Troubleshoot..... | 39 |

Overview


The MD330U Series PTZ Camera is a dual-mode medical grade camera designed with a detachable lens. It comes in two models: MD330U (fill light) and MD330UI (infrared night vision).

Note: Product design may vary by region. Please refer to your current device and consult your local sales representative or technical support for specific details.

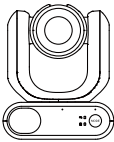Package Contents


Camera

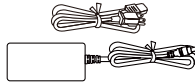
Power Adapter &
Power Cord

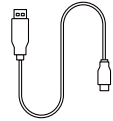
Remote Control

USB 3.0 Type-A to
Type-B Cable (1.5m)

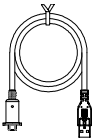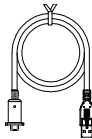Pre-Installed
Lens Cable**
(standard, 180 cm)

Lens Cable**
(standard short, 45 cm)

1/4\"-20,
L=7.5mm (x2)

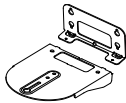
Quick Start Guide


Optional Accessories

Lens Cable**
(long, 600 cm)

Wall Mount Bracket

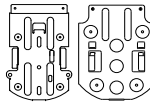Ceiling Mount Bracket
(x2)

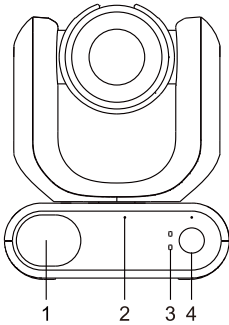
Camera Controller
(CL01)


* For detail on optional accessories, consult your local dealer.

**For lens cable changing instructions, refer to the user manual.


Parts Info

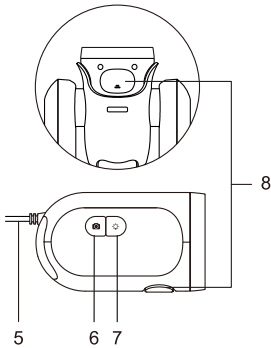1. Speaker
2. Microphone Array
3. LED Indicators (Standard Mode / Handheld Mode)
4. Mode Button

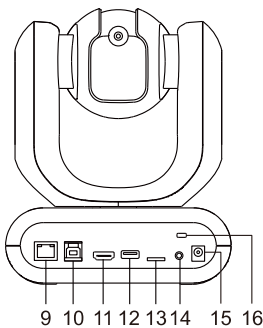5. Pre-Installed Lens Cable (Standard, 180 cm)
6. Snapshot Button
7. Fill Light Button (MD330U only)
8. Latch

9. Ethernet Port
10. USB 3.0 Type-B Port
11. HDMI® Port
12. Lens Cable Port
13. microSD Slot (32GB/SDXC)
14. RS-232 Port
15. DC 12V Power Jack
16. Kensington Lock

LED Indicators

The illumination of the indicators confirms the camera mode.

Standard Mode
(Default)


Handheld Mode


| Color | Status | Pattern |
|--|-------------------|---------|
| Flashing orange | Start-up | ●● |
| Solid blue | Normal | ● |
| Solid orange | Standby | ● |
| Flashing orange slowly (1-sec interval) | Muted | ● |
| Flashing orange (0.5-sec interval) | Recording | ⊙ |
| Flashing orange once | Taking a snapshot | ⊙ |
| Solid purple | OSD menu on | ● |
| Flashing purple | Firmware update | ●● |

- Both indicators flash.
- The indicator is mode-sensitive.
- ⊙ The spare indicator flashes.

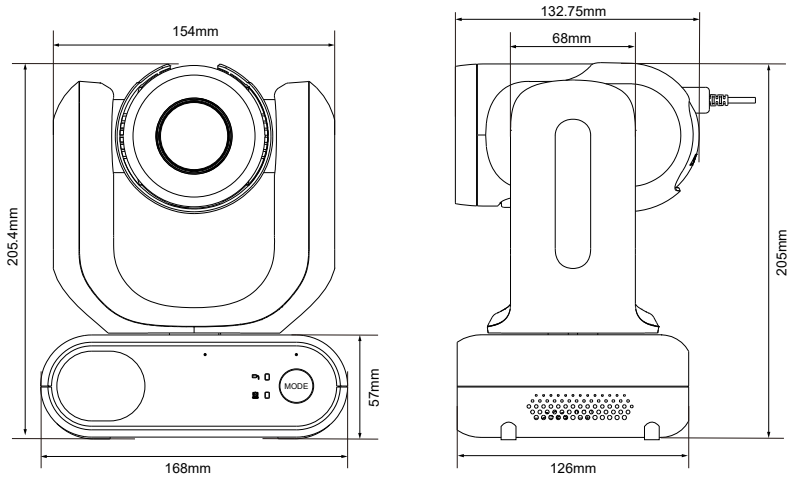Dimensions

Pan and Tilt Angle

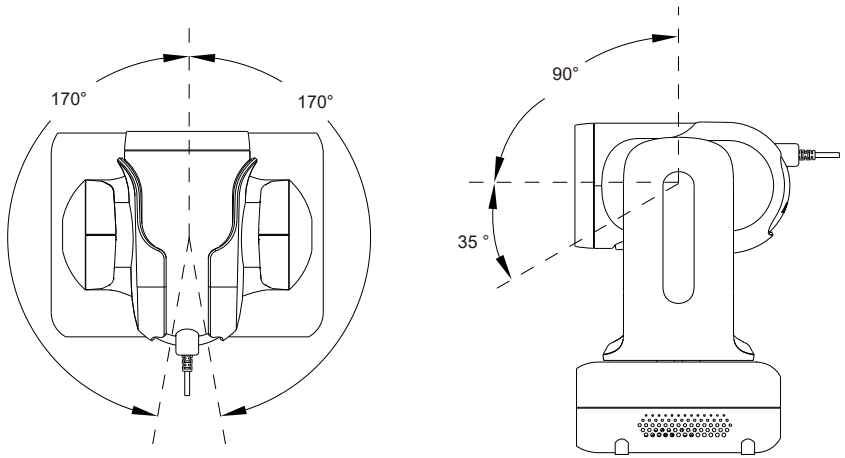
Change Your Lens Cable

We offer lens cable in 3 different lengths:

- Standard cable: 180cm pre-installed
- Standard short cable: 45cm in the package
- Long cable: 6m optional accessory

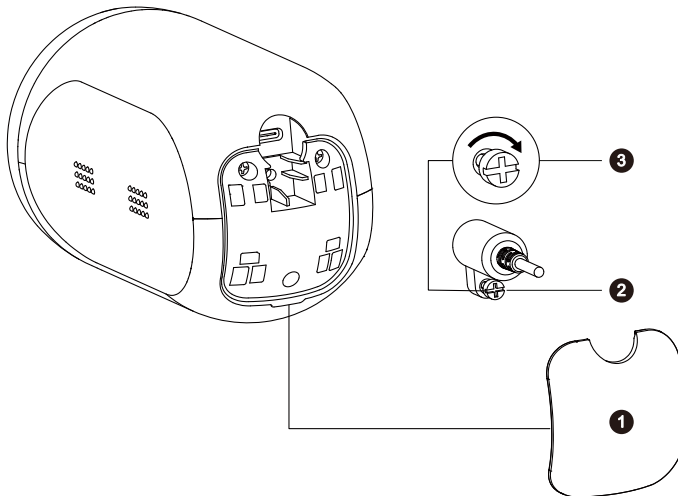Switch between Standard and Standard Short Cable

Note:

- The standard cable is pre-installed. If Handheld Mode isn't required, you can replace the standard cable with the standard short cable.
- **Fully tighten the cable screw** as loose contact results in failure to power on or flickering image.

1. Disconnect power from the camera. Remove the back cover by gently pulling up from the notch on the bottom.
2. Loosen the cable screw and unplug the cable.
3. Plug in the replacement cable, tighten the screw, and close the back cover.

Switch from or to a Long Cable

Note:

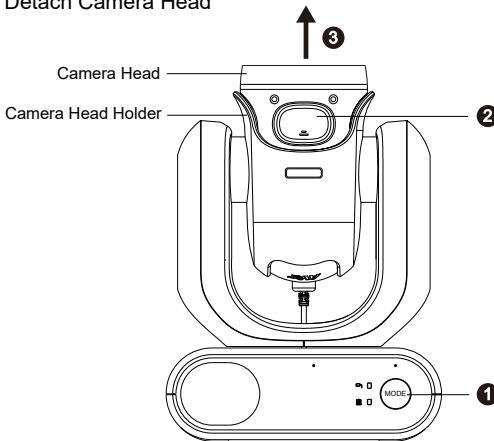
- If you are switching from or to a long cable, make sure to **enable the correct lens cable mode** on the web interface or the OSD menu. Switching from or to a long cable without enabling the correct mode may cause the device to display a black screen or flickering image.
- The lens cable mode setting reverts to **Standard cable** after a factory reset.
- **Fully tighten the cable screw** as loose contact results in failure to power on or flickering image.

1. Connect power to the camera. Enable the correct lens cable mode based on the lens cable you want to use:
 - Web interface: Go to **Systems > Change cable > select Standard cable or Long cable.**
 - OSD menu: Go to **Systems > Change cable > select Standard cable or Long cable.**
2. When prompted to reboot, disconnect power from the camera.
3. Remove the back cover by gently pulling up from the notch on the bottom.
4. Loosen the cable screw and unplug the cable.
5. Plug in the replacement cable, tighten the screw, and close the back cover.
6. Connect power to the camera.


Detach or Install the Lens

The lens must be detached or installed in Handheld Mode. Press the mode button to switch between Standard Mode and Handheld Mode, then use the LED indicators to confirm your current mode.

Detach Camera Head

Install Camera Head

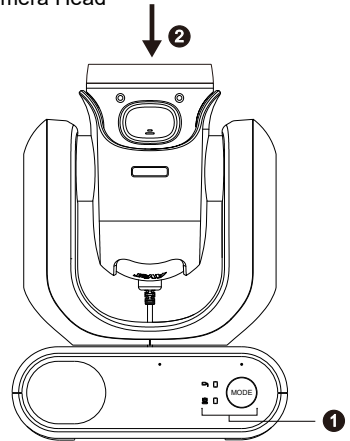To detach the lens:

1. Press the mode button to switch from Standard Mode to Handheld Mode. The lens will automatically turn up.
2. Press the latch.
3. Remove the lens from the lens holder.

To install the lens:

1. Make sure the camera is in Handheld Mode. If not, press the mode button to switch from Standard Mode to Handheld Mode.
2. Insert the lens in the lens holder until you hear a click.

To take snapshots in Handheld Mode:

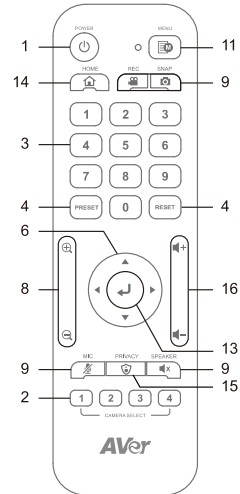
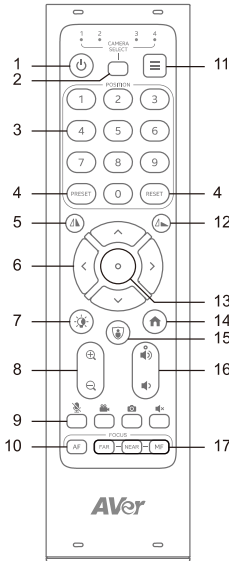
- Make sure a microSD card has been inserted in the slot.
- Select a focus distance on the web interface (see [Camera Control](#) > Focus Near Limit).
- Press and hold the snapshot button for 1 second to take a snapshot. Press the snapshot button once to autofocus.

To turn on the fill light in Handheld Mode (MD330U):

1. Press the fill light button on the lens to turn it on, and press the button repeatedly to cycle through 3 brightness levels (On > Low > Medium > High > Off).
2. With the fill light on, the closest shooting distance should be 4.5 cm.

Remote Control

- To open the OSD menu, press and hold **Menu**  for 3 seconds.
- To disable remote control, open the OSD menu or the web interface, go to **System > Camera Selector > Disable Remote**.
- To resume remote control, open the web interface, go to **System > Camera Selector > All Channel** or assign a number (**1, 2, 3, 4**) to your camera.
- Your device may come with one of the following remote controls. Update to the latest firmware for buttons with an asterisk (*).


| Name | Function |
|---------------------|---|
| 1. Power | Enter Standby Mode or wake up. |
| 2. Camera Select | No selection is required to operate the camera by default. <ul style="list-style-type: none"> The remote is set to channel 1, and the camera responds to all channels at the factory. To assign a number to the camera, go to System > Camera Selector in the OSD menu. |
| 3. Number Buttons | Press Number button (0-9) to load defined preset 0-9. |
| 4. Preset/Reset | <ul style="list-style-type: none"> To save a preset, press and hold Preset, then a Number button (0-9). To reset a preset, press and hold Reset, then press a Number Buttons (0-9). |
| 5. Mirror* | Mirror the image. |
| 6. Pan-Tilt Control | Pan and tilt direction control. |

| | |
|-------------------------------|--|
| 7. Fill Light / IR* | Turn fill light / infrared on or off. |
| 8. Zoom In/Out | Zoom in or out. |
| 9. Function Buttons | <ul style="list-style-type: none"> • Mic: Mute or unmute the microphone. • Rec: Start or stop recording. • Snap: Take a snapshot. • Speaker: Mute or unmute the speaker. |
| 10. Auto Focus* | Turn Auto Focus on or off. |
| 11. Menu* | <p>Press and hold for 3 seconds to open the OSD menu.</p> <ul style="list-style-type: none"> • To change to a short press, go to System > Trigger OSD in the OSD menu. |
| 12. Flip* | Flip the image. |
| 13. Enter | <ul style="list-style-type: none"> • Confirm a selection in the OSD menu. • Press to auto focus once. |
| 14. Home | Reset the pan-tilt position to the center. |
| 15. Privacy | Enter or exit Privacy mode (The camera points downward, stops video, and mutes the microphone.). |
| 16. Volume +/- | Volume up or down. |
| 17. Far / Near / Manual Focus | Press Manual Focus to turn on Manual Focus first, then press Far or Near to adjust focus manually. |



Shortcuts

| Press | To |
|--|---|
| Power  for 10 seconds | Reboot. |
| Menu  for 3 seconds | Open the OSD menu. |
| Menu  | Close the OSD menu. |
| Home  | Close the OSD menu and reset the pan-tilt position to the center. |
| Menu  then Zoom In  | MD330U: Cycle through 3 brightness levels (On > Low > Medium > High > Off). MD330UI: Switch between Day mode and Night mode. |
| Menu  then Zoom Out  | MD330UI: Switch between Day mode and Auto mode. |
| 5 five times (55555) <i>Firmware 1.1.1005.1 or later</i> | Turn on DHCP. |
| 6 six times (666666) <i>Firmware 1.1.1030.0 or later</i> | Reset the camera to factory default settings. |
| 8 eight times (88888888) <i>Firmware 1.1.1005.1 or later</i> | Set the camera's static IP address to 192.168.1.168. |

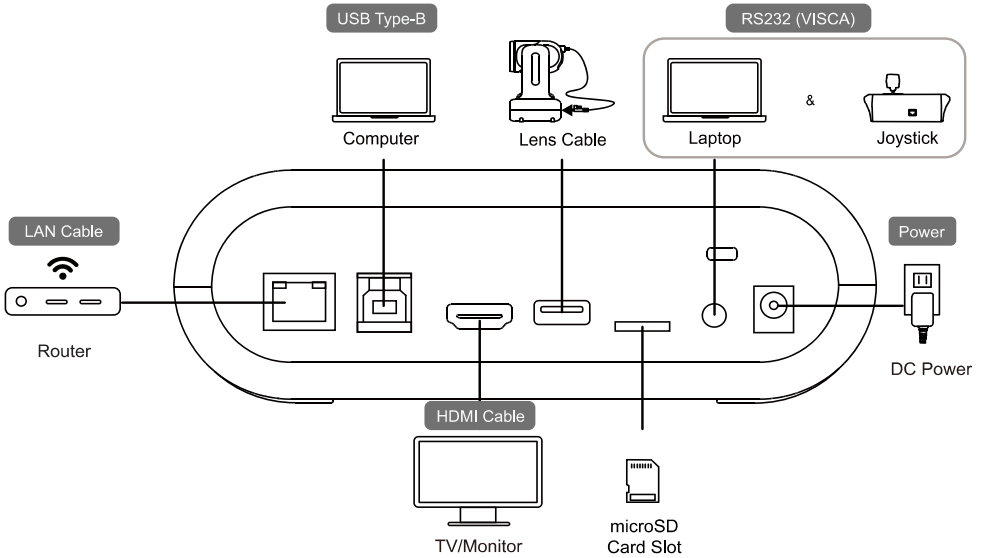Connections


| Port | Description |
|-----------------|---|
| Ethernet port | Connect to a router to access the device's web interface. |
| USB Type-B port | Connect to a computer to transmit video when using a video conferencing software. |
| HDMI port | Connect to a TV or monitor with a grounding design. |
| USB Type-A port | Connect the lens cable to the lens cable port on the bottom of the device and tighten the lens cable screw to secure. |
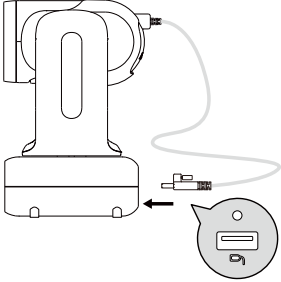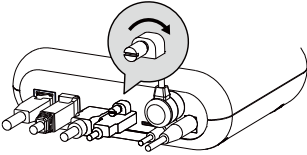
| | | |
|-------------------|--|-----------------------------------|
| microSD card slot | Insert a microSD card to store images or videos. Compatible microSD cards: | |
| | Brand | Kingston, SanDisk, Samsung, Lexar |
| | Speed Class | C10, U1, U3, V10, V30 |
| | Capacity | 16G, 32G |
| | Format | FAT32, exFAT |
| RS-232 port | Connect to a joystick or computer to control the camera. You can optionally purchase the RS-232 adapter. | |
| DC power jack | Connect to a power outlet using the supplied power adapter and power cord. | |

RS-232 Pin Definition

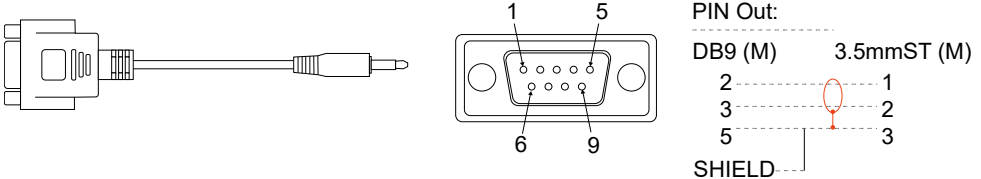RS-232 Adapter (3.5mm to D-Sub 9-pin)

Note:

Accessory equipment connected to the analog and digital interfaces must be in compliance with the respective nationally harmonized IEC standards (i.e. IEC 60950 for data processing equipment, IEC 60065 for video equipment, IEC 61010-1 for laboratory equipment, and IEC 60601-1 for medical equipment.) Furthermore all configurations shall comply with the requirements of the system in standard IEC 60601-1. Everybody who connects additional equipment to the signal input part or signal output part configures a medical system, and is therefore, responsible that the system complies with the requirements of the system in standard IEC 60601-1. The device is for exclusive interconnection with IEC 60601-1 certified equipment in the patient environment and IEC 60XXX certified equipment outside of the patient environment. If in doubt, consult the technical services department or your local representative.

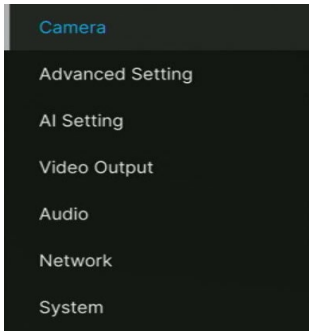
Set Up Your Camera

You can configure camera settings using the OSD menu or the web interface of the camera.

OSD Menu

To access the OSD menu, connect the camera to a monitor or TV using an HDMI cable, and then use the supplied remote control to operate the OSD menu.

Press and hold the menu button  for 3 seconds to bring up the OSD menu. Use directional buttons     to select and press Enter  to confirm settings.

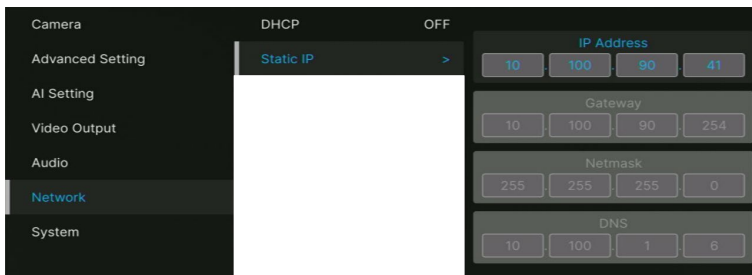
IP Address Setup

Static IP

1. Press the menu button  on the remote control to bring up the OSD menu.
2. Go to **Network > Static IP**.

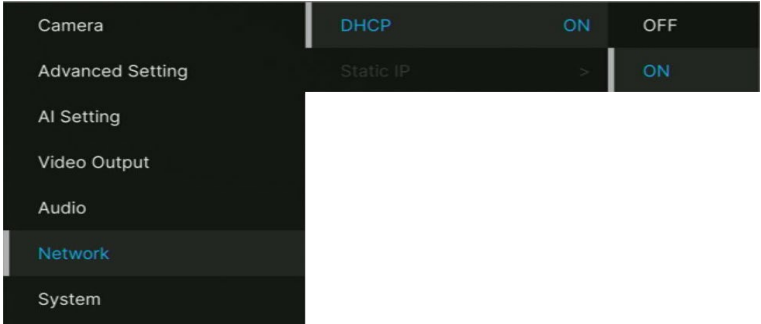
[Note] Turn the **DHCP** off before setting up static IP (**Network > DHCP > OFF**).

3. Select the **IP Address**, **Gateway**, **Netmask**, and **DNS** to configure. Press Enter  and use the number buttons to enter the value.


DHCP

1. Press the menu button  on the remote control to bring-up the OSD menu.
2. Select **Network** > **DHCP** > **On**.
3. Press Enter  to confirm setting.

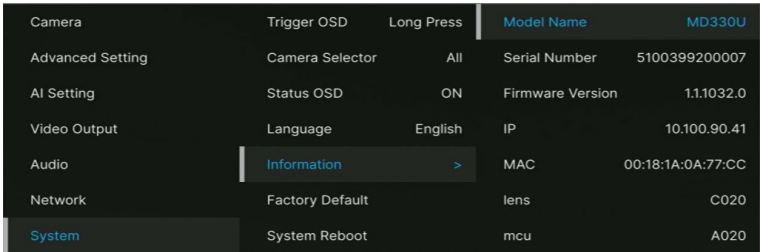4. After turning the **DHCP** on, you can go to **System** > **Information** to view the IP address.

OSD Menu Tree

| 1 st Level | 2 nd Level | 3 rd Level | 4 th Level |
|---------------------------|-----------------------|-----------------------|--------------------------|
| Camera | Exposure Mode | Full Auto | Exposure Value |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | | WDR |
| | | Shutter Priority | Exposure Value |
| | | | Shutter Speed |
| | | | Gain Limit Level |
| | | | BLC |
| | | Iris Priority | Exposure Value |
| | | | Iris Level |
| | | | Gain Limit Level |
| | Slow Shutter | | |
| | BLC | | |
| | Manual | Iris Level | |
| | | Shutter Speed | |
| | | Gain Level | |
| | White Balance | Auto | |
| | | ATW | |
| | | Indoor | |
| | | Outdoor | |
| | | One push | |
| | | Manual | R gain |
| | | B gain | |
| | Pan Tilt Zoom | Preset Speed | 5, 25, 50, 100, 150, 200 |
| | | Preset Accuracy | OFF / ON |
| | | Sensor Zoom | OFF / ON |
| | | Sensor Zoom Limit | x2 |
| Pan Speed | | 1~24 | |
| Tilt Speed | | 1~24 | |
| Zoom Speed | | Low / High | |
| P/T Spd. Relative Z Ratio | | OFF / ON | |

| | | | |
|------------------|-----------------------|---------------------------|-----------------------------|
| | | Pan L/R Dir. Switch | OFF / ON |
| | | Focus Mode | Manual / Auto |
| | Noise filter | Off / Low / Middle / High | |
| | Saturation | 0 1 2 3 4 5 6 7 8 9 10 | |
| | Contrast | 0 1 2 3 4 | |
| | Sharpness | 0 1 2 3 | |
| | Mirror | OFF / ON | |
| | Flip | OFF / ON | |
| Advanced Setting | Control | Type | RS232 |
| | | Protocol | VISCA / PELCO D/PELCO P |
| | | Camera Address | 1 2 3 4 5 6 7 |
| | | Baud Rate | 2400 / 4800 / 9600 / 115200 |
| | IR Cut Filter | Auto / Day / Night | |
| | IR Cut Sensitivity | Low / Middle / High | |
| | Performance Mode | OFF / ON | |
| | AI Mosaic | OFF / NET / HDMI | |
| Mosaic Type | Body / Full Face | | |
| AI Setting | Eyes Tracking | Eyes Tracking | OFF / ON |
| | | Tracking Site | Save |
| | | Tracking Type | Full Face/Eyes |
| | | Tracking Range | Close / Medium / Wide |
| | | Timeout to Site | 3/5/7/10 sec |
| | Detection | Detection | OFF / ON |
| | | Detection Type | Body / Full Face |
| | Fall/Egress Detection | Fall/Egress Detection | OFF / ON |
| Detection Type | | All Type / Fall / Egress | |
| Video Output | Frequency | 60 | |
| | | 59.94 | |
| | | 50 | |
| | Resolution | 1080p60 | |
| | | 1080p59.94 | |
| | | 1080p30 | |
| | | 1080p29.97 | |
| | | 720p60 | |
| 720p59.94 | | | |

| | | | | |
|------------|------------------|---|-----------------------------------|--|
| | | 1080p50 | | |
| | | 1080p25 | | |
| | | 720p50 | | |
| | | 720p30 | | |
| Audio | Audio Input | Noise Suppression | OFF/ NR DSP/ NR/ NR+BF/ NR Strong | |
| | | Audio Beam Forming | 30,50,70,90,110,130 | |
| | | Audio In Volume | Mute / 1~10 | |
| | Audio Output | Output Type | Speaker/HDMI/Both | |
| | | Output Volume | Mute / 1~10 | |
| Network | DHCP | OFF / ON | | |
| | Static IP | IP Address | 192.168.1.168 | |
| | | Gateway | 192.168.1.254 | |
| | | Mask | 255.255.255.0 | |
| | | DNS | 8.8.8.8 | |
| System | Trigger OSD | Click Menu to open, Press Menu 3 sec | | |
| | Camera Selector | 1,2,3,4, All channel, Disable Remote | | |
| | Status OSD | OFF / ON | | |
| | Language | English / 中文 / 日本語 | | |
| | Change cable | Standard cable / Long cable | | |
| | Information | Model Name | MD330U(I) | |
| | | Serial number | xxxxxxxxxxxx | |
| | | Firmware Version | 0.0.0000.00 | |
| | | IP | 192.168.1.168 | |
| | | MAC | 00:18:1a:04:9e:81 | |
| | | lens | xxxx | |
| | | mcu | xxxx | |
| | | SD Total (MB) | | |
| | | SD Free (MB) | | |
| | Recording Status | Stopped / xx:xx:xx (record timer) | | |
| | Factory Default | OFF / ON | | |
| | System Reboot | OFF / ON | | |
| Theme Mode | AVer/Zoom | | | |

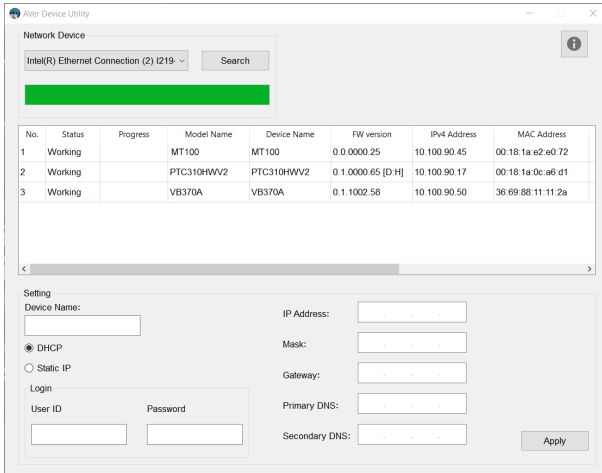
Access the Web Interface

To access the web interface of the camera, you can use any of the following software to find its IP address:

- AVer Device Utility
- AVer ViewCare

Note: The camera default network is Static IP (192.168.1.168).

AVer Device Utility

To access the web interface:

1. Download AVer Device Utility from AVer Download Center (<https://www.aver.com/download-center>) and launch the software.
2. Click **Search** to see available devices on the same local area network (LAN).

Note:

- Make sure your device is connected to the internet.
 - AVer Device Utility and your device must be on the same LAN.
3. Double-click on your device's IP address in the **IPv4 Address** column to open the web interface in your browser.

When you log in for the first time:

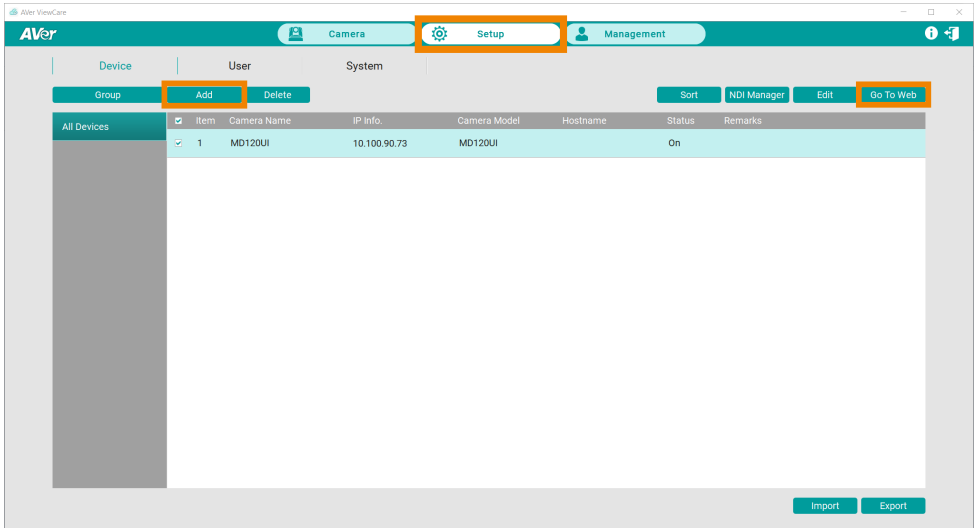
Change the username and password before logging in to the web interface.

- Username: Use 1-32 characters.
- Password: Use 8-32 characters and a combination of uppercase letters, lowercase letters, numbers, and symbols (%+=, -_ ^/ @. ~). The password cannot be the same as the username.

To change your network to DHCP or static IP:

1. Select the checkbox of your camera.
2. Enter the changed username and password in the **Login** field.
3. Select **DHCP** or **Static IP**, then enter your network settings if applicable in the **Settings** section.
4. Click **Apply**.

AVer ViewCare


Note: The AVer ViewCare default username and password is **admin/admin**.


1. Download AVer ViewCare from AVer Download Center (<https://www.aver.com/download-center>) and launch the software.
2. Log in with the AVer ViewCare default username and password **admin/admin**. Then enter your email address for password recovery.
3. Go to **Setup > Add**, then click **Auto Search** to see available devices on the same local area network (LAN).
4. Click to select your camera, enter the changed camera username and password, then click **Save** to add the camera to the device list.
5. Select the checkbox of your camera, then click **Go to Web** button to open the web interface in your browser.

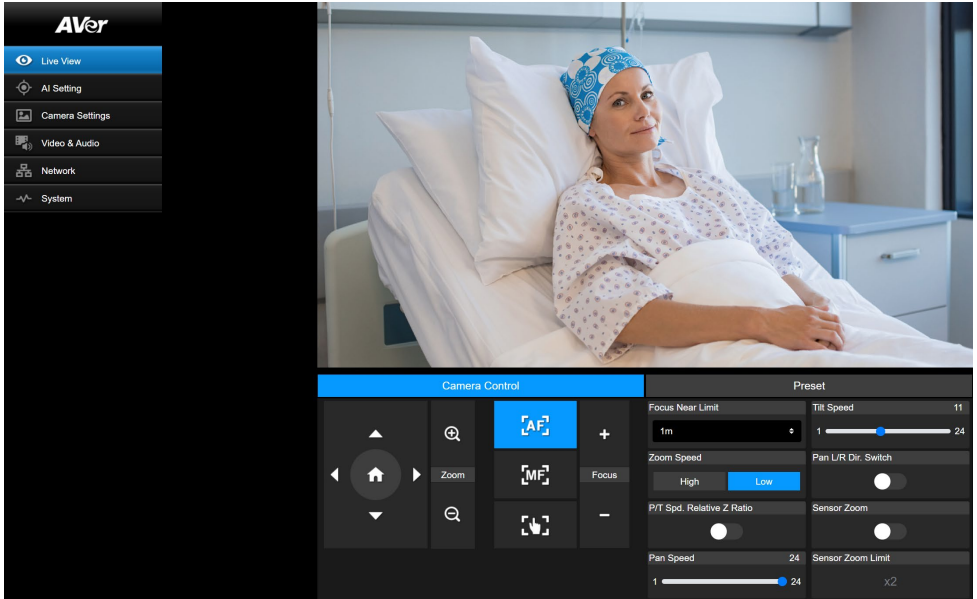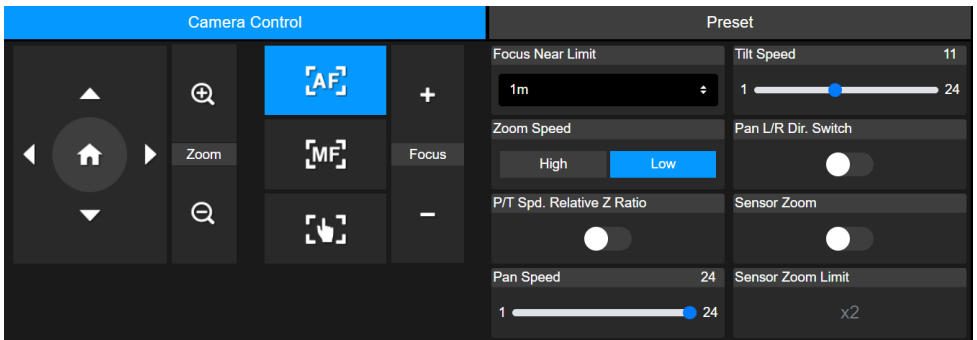Web Interface

Live View


Camera Control

| Item | Description |
|-----------------------|--|
| Pan and Tilt Controls | Position the camera. <ul style="list-style-type: none">• Drag the slider to adjust Pan Speed and Tilt Speed.• Turn on Pan L/R Dir. Switch to invert panning direction. |

| | |
|--|---|
| | <ul style="list-style-type: none"> Turn on Pan Spd. Relative Z Ratio automatically adjust pan and tilt speeds based on the zoom ratio. |
| Home Position  | Move the camera to the Home position. |
| Zoom   | <ul style="list-style-type: none"> Zoom in or zoom out the live view and select Zoom Speed. Turn on Sensor Zoom to enable sensor zoom. Up to 30X optical zoom and 2X sensor zoom (Sensor Zoom Limit). |
| Focus +/- | <ul style="list-style-type: none">  Auto Focus: Click to automatically focus.  Manual Focus: Click to manually focus. Adjust the focus with +/- buttons.  One Push Focus: Click to automatically focus once. Focus Near Limit: Set up the nearest focus limit. |

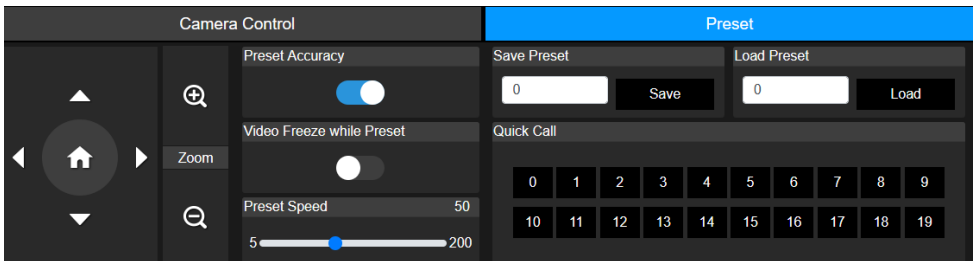Preset

| Item | Description |
|---------------------------|---|
| Save Preset | <ol style="list-style-type: none"> Position the camera using pan, tilt and zoom controls. Enter a preset number (0~255) in the Save Preset field and click Save. |
| Load Preset | <ol style="list-style-type: none"> Enter a preset number (0~255) in the Load Preset field and click Load. Or click a preset number (0~19) in the Quick Call section. |
| Preset Accuracy | Turn on to improve the accuracy of moving to presets. |
| Video Freeze while Preset | Turn on to display only the live view from presets. The live view from the moving path will not be displayed. |
| Preset Speed | Adjust the camera speed when moving to presets. |

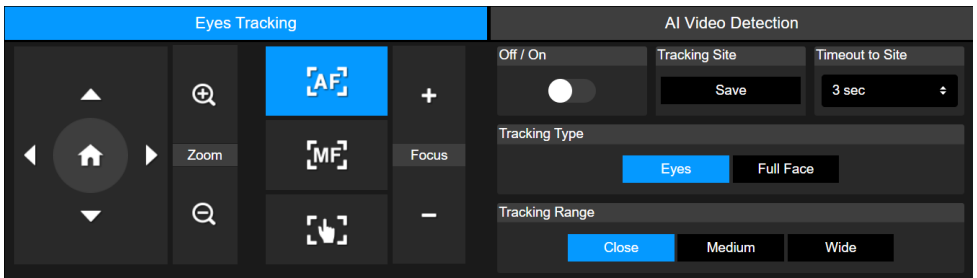
AI Setting

Detection and Tracking can't be turned on at the same time.

Eyes Tracking

Manually adjusting pan, tilt and zoom controls during Eyes Tracking will turn off the function.



1. Position the camera using pan, tilt and zoom controls and then click **Save** to save a **Tracking Site**. The default is the Home position.
2. From the **Timeout to Site** drop-down list, select an interval before the camera returns to the tracking site when no one is on screen. The default is 3 seconds.
3. Select **Tracking Type** and **Tracking Range**.

4. Toggle on eyes tracking.

- One person: The camera automatically tracks and zooms in on the face.
- Multiple people: The camera frames all faces in bounding boxes. Click a square to select a face you want to track and zoom in on.

AI Video Detection

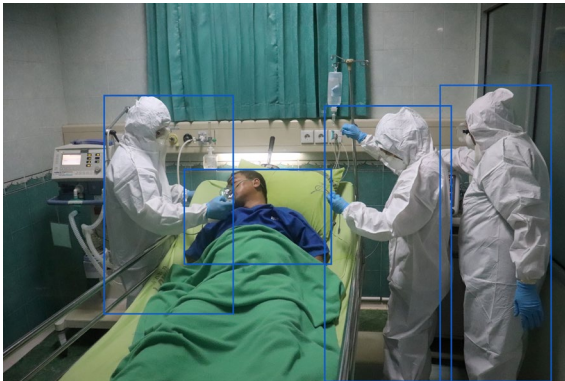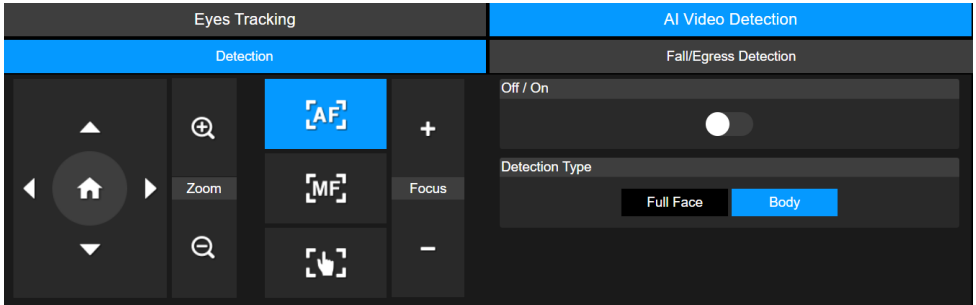
- **Detection**

Click the **On/Off** toggle to turn on face or body detection.

- The camera will frame faces or bodies on screen with blue AI bounding boxes.
- Use API to report AI bounding box coordinates to third-party software.

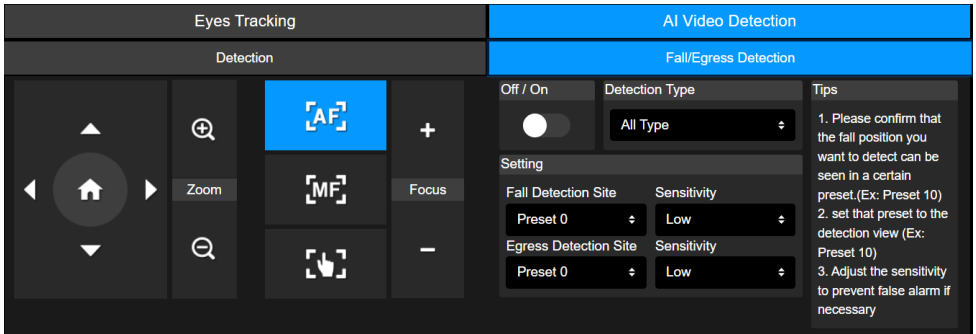- **Fall/Egress Detection**

To set up fall or egress detection:

1. Make sure you have defined the required preset where a fall or egress event may occur.
2. Select that preset from the **Fall Detection Site** or **Egress Detection Site** drop-down list, and select the detection sensitivity.
3. Select the **Detection Type**.
4. Toggle on AI video detection.

DISCLAIMER

Learn about the accuracy and limitations of the AI video detection.

The AI video detection is an advanced technology designed to enhance patient safety in hospital environments.

Limited Accuracy

The ability of the AI video detection to accurately identify a fall or egress event was tested in a controlled environment. This does not guarantee the successful identification and alarms for all fall or egress events in real world use. Many factors can affect the performance of the AI video detection, such as an obstructed field of view or restricted view angle.

Dose not Replace Caregivers

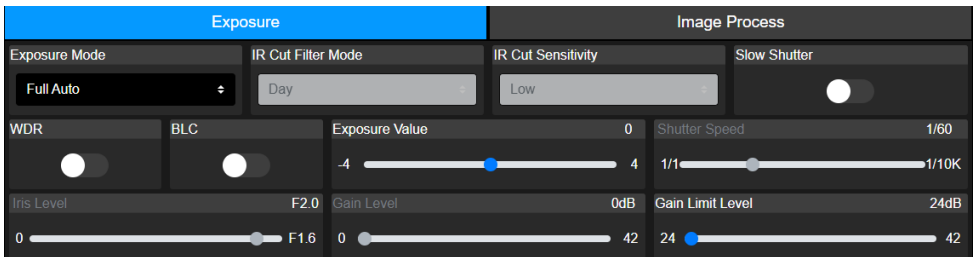
The AI video detection is intended as an assistive tool and must not be used as the replacement for caregivers. Frameworks which mandate periodic checks by a trained professional to ensure patient safety and which relate to clinical diagnosis, patient care or treatment must remain in place. If AI video detection fails to respond as described, take appropriate action immediately.

The AI video detection is intended as an assistive tool and is not part of a medical device. It is not a patient monitor and cannot be used to make a medical or clinical decision.

Camera Settings


Exposure

| Item | Description |
|------------------|--|
| Exposure Mode | Choose an exposure mode. |
| WDR | Turn wide dynamic range (WDR) or backlight compensation (BLC) on or off. |
| BLC | |
| Exposure Value | Adjust exposure, shutter, iris and gain. |
| Shutter Speed | |
| Iris Level | |
| Gain Level | |
| Gain Limit Level | |


| | |
|--------------------------------------|--|
| IR Cut Filter Mode (MD330UI only) | Choose Day mode or Night mode to switch infrared night vision on or off. Or select Auto mode and adjust IR Cut Sensitivity . |
| Slow Shutter | Turn Slow Shutter on or off. |

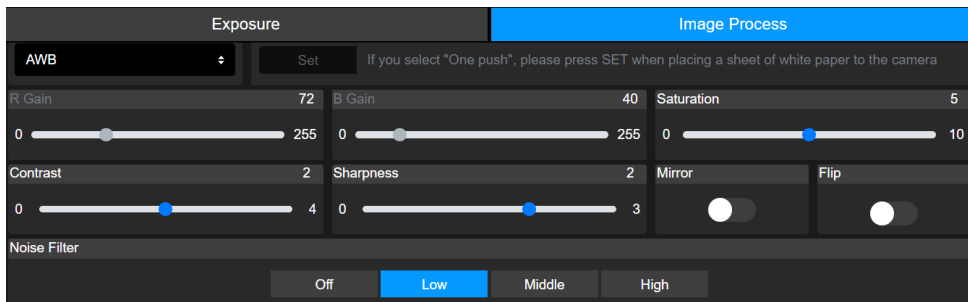Image Process

| Item | Description |
|---------------|--|
| White Balance | <ul style="list-style-type: none"> Select a white balance mode. When Manual is selected, you can also adjust the R Gain and B Gain. When One Push is selected, place a piece of white paper in front of the camera lens and click Set to calibrate white balance. |
| Saturation | Adjust saturation, contrast and sharpness. |
| Contrast | |
| Sharpness | |
| Mirror | Turn Mirror or Flip on or off. |
| Flip | |
| Noise Filter | Select a noise filtering level. |

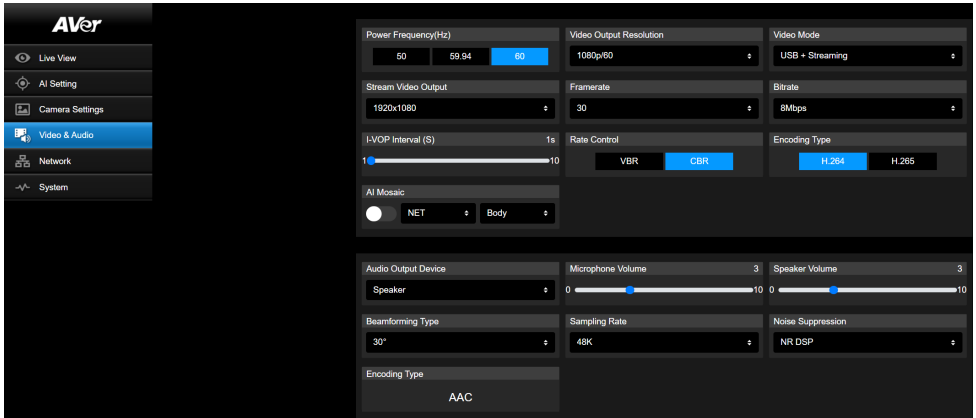Video & Audio

Video Setting


| Item | Description |
|----------------------|--|
| Power Frequency (Hz) | Select 50Hz , 59.94Hz or 60Hz based on your country or region. |
| Video Out Resolution | Select a video output resolution. RTSP: Max. 4K/30fps HDMI: Max. 1080p 60Hz |
| Video Mode | Select a video mode. <ul style="list-style-type: none"> • USB only: Stream video over USB. • Stream only: Stream video over IP. • USB + Streaming: Stream video over both USB and IP. |
| Stream Video Output | Select a streaming output resolution for the live view. |
| Framerate | Select a framerate |
| Bitrate | Select a bitrate. |
| I-VOP Interval (S) | <ul style="list-style-type: none"> • Drag the slider to choose how often I-VOPs appear in a video stream. • Shorter I-VOP intervals result in higher video quality but also larger file sizes. |
| Rate Control | Select VBR or CBR . |
| Encoding Type | Select H.264 or H.265 . |
| AI Mosaic | Turn on to pixelate the face or body for privacy on an IP or HDMI stream. |

Audio Setting


| Item | Description |
|---------------------|--|
| Audio Output Device | Select an audio source. |
| Microphone Volume | Drag the slider to adjust the microphone volume. |
| Speaker Volume | Drag the slider to adjust the speaker volume. |
| Beamforming Type | Set the Noise Suppression to NR + BF , then select a beamforming angle. Refer to <Beamforming Angles> for details. |
| Sampling Rate | Select an AAC sampling rate for IP streaming. |
| Noise Suppression | Select a noise suppression setting. Start with NR before trying other options closer to your use case. Refer to <Noise Suppression Performance> for details. |
| Encoding Type | AAC |


Beamforming Angles

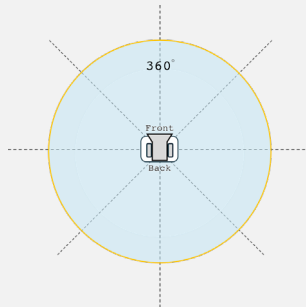Note: The device uses two (front and back) omnidirectional (360°) microphones with algorithm-based beamforming. While pickup is focused within the selected angle, loud sounds outside this range may still be picked up in noisy environments.

Noise Suppression Setting: NR + BF

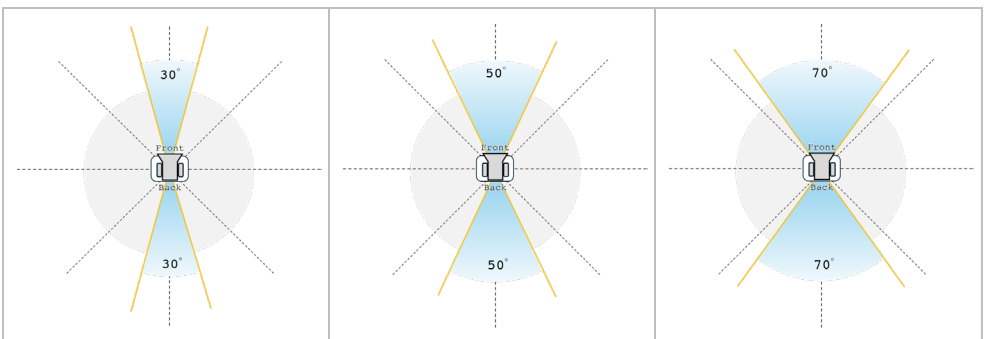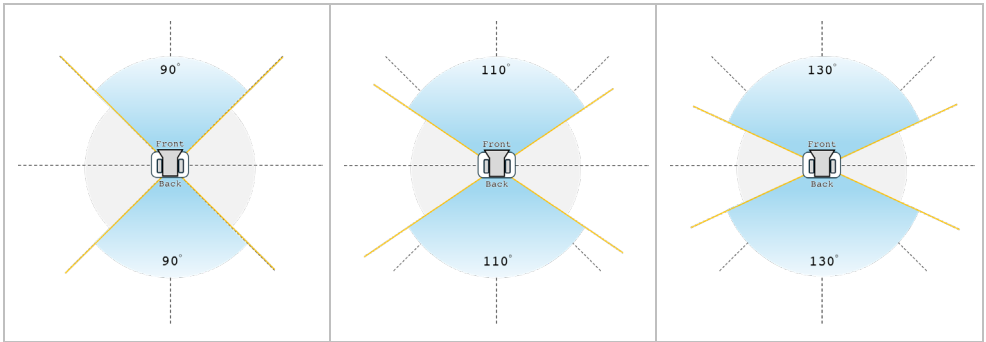
Noise Suppression Performance

| Noise Suppression Settings | Use Case | Performance* |
|----------------------------|---|---|
| OFF | Very quiet indoor environment. | Noise is kept. |
| NR DSP | Indoor environment with low noise. | Noise is reduced |
| NR (recommended) | Indoor environment with strong background noise, such as a fan. | Noise is further reduced |
| NR+BF | When it is necessary to limit the range of sound collection with specified angle. | <ul style="list-style-type: none"> ✓ Noise is removed ✗ Might drop a few words |
| NR Strong | Indoor environment with strong background noise, such as people talking. | <ul style="list-style-type: none"> ✓ Noise is removed ✗ Takes longer to process and reduce unusual sounds ✗ Might drop a few words |

*Performance varies due to many factors, including voice to noise ratio, number of talkers, room size, sound source direction, microphone quality, ambient sounds, and so on. Performance was tested under controlled AVer laboratory conditions.

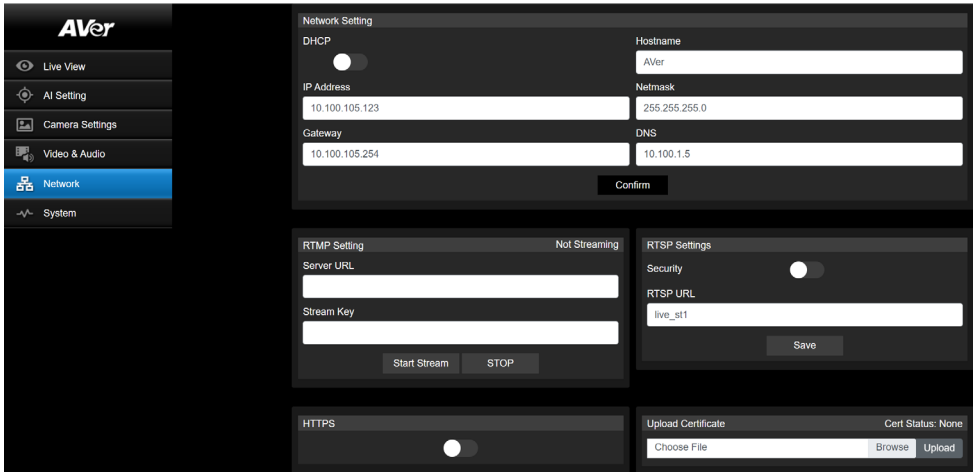
Network

| Item | Description |
|---------------|--|
| Hostname | The default hostname is AVer. Change the hostname to be displayed on devices such as an IP router. |
| DHCP | Set the network to DHCP or Static IP. <ul style="list-style-type: none"> • DHCP: Turn on DHCP and click Confirm to save the setting. The camera will be assigned IP settings automatically. • Static IP: Turn off DHCP, enter IP Address, Netmask, Gateway and DNS, and click Confirm to save the settings. |
| RTMP Setting | Stream live video to a video platform such as YouTube. <ol style="list-style-type: none"> 1. Enter the Server URL and Stream Key of your video platform. Please refer to the instruction of your platform to obtain the server URL and stream key. 2. Click Start Stream to start streaming, Stop to stop streaming. |
| RTSP Settings | Protect your video stream on media players such as VLC, PotPlayer and QuickTime by ensuring that only authorized users can access it. <ul style="list-style-type: none"> • When Security is turned off: <ol style="list-style-type: none"> 1. Enter your camera's RTSP URL into the media player. 2. RTSP URL: rtsp://[camera IP address]/live_st1 Example: rtsp://192.168.1.100/live_st1 • When Security is turned on: <ol style="list-style-type: none"> 1. Enter your camera's RTSP URL and username/password into the media player. 2. RTSP URL: rtsp://[username:password]@[camera IP address]/live_st1 |

| | |
|-------|--|
| | <p>Example: rtsp://1:1@192.168.1.100/live_st1</p> <p>3. username/password: camera's username/password (web interface login)</p> |
| HTTPS | <p>Enable HTTPS to establish a secure connection between your browser and your camera. To enable HTTPS access on your camera:</p> <ol style="list-style-type: none"> 1. Obtain a SSL certificate for encryption and decryption in base-64 encoded format and use a private key in PKCS#8 format (unencrypted). 2. Package the required certificate content into PEM format. The SSL certificate uploaded to the camera must be in PEM format. 3. Click Browse to select the certificate file, and then click Upload. 4. Turn on HTTPS. |

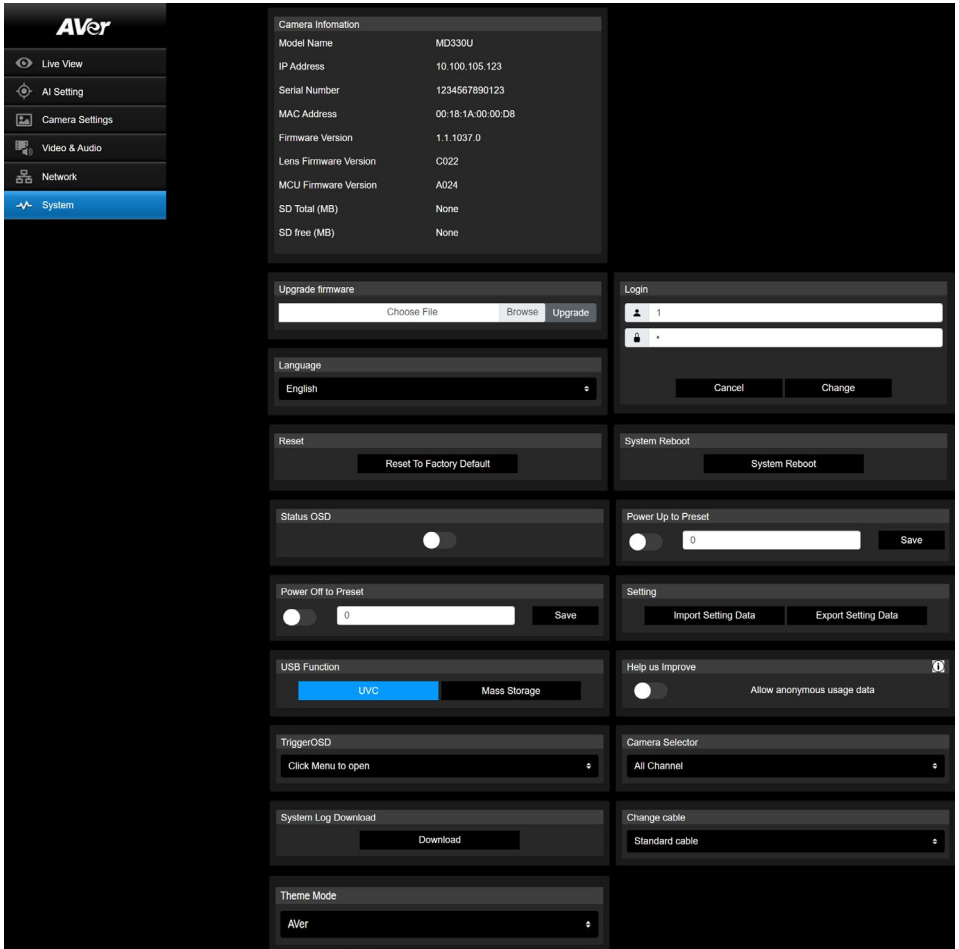System

| Item | Description |
|--------------------|--|
| Camera Information | Display camera information. |
| Upgrade Firmware | <p>Follow these steps to upgrade the firmware:</p> <ol style="list-style-type: none"> 1. Download the latest firmware from the AVer Download Center (https://www.aver.com/download-center/). 2. On the Web page, go to System > Upgrade firmware. 3. Click Browse to select the firmware. 4. Click Upgrade to start upgrading the firmware. 5. Refresh the browser after the upgrade process is complete. |

| | |
|---------------------|---|
| | <p>Note: Keep your camera connected to a power source during firmware upgrade. Network connection will be lost during the process and camera will reboot automatically after upgrading.</p> |
| Login | The default username and password is admin/admin . To change the username or password, enter the new name or password and click Change . |
| Language | Change the web interface language. |
| Reset | Reset the camera to factory default settings. <ul style="list-style-type: none"> Resetting while in Zoom Mode will load Zoom Mode settings. To reset to AVer factory default settings, go to System > Theme Mode > select AVer. |
| System Reboot | Restart your camera. |
| Status OSD | Enable to display preset status on the HDMI output during functions such as saving, loading and canceling presets. |
| Power Up to Preset | When enabled, the camera will move to the defined position after powering on. <ul style="list-style-type: none"> Toggle on > enter a preset number > click Save. Make sure the preset number has been defined before enabling this function. |
| Power Off to Preset | When enabled, the camera will move to the defined position before powering off. <ul style="list-style-type: none"> Toggle on > enter a preset number > click Save. Make sure the preset number has been defined before enabling this function. |
| Setting | Import or export your camera settings. |
| USB Function | Select UVC or Mass Storage . <ul style="list-style-type: none"> UVC: Stream video over USB. Mass Storage: Access the inserted microSD card via USB connection. <ol style="list-style-type: none"> Connect the camera to a computer using the included USB 3.0 Type-B to Type-A cable. Make sure a microSD card has been inserted in the slot. On the web interface, go to System > USB Function > select Mass Storage. |
| Help Us Improve | Opt-in or opt-out of providing anonymous usage data. |
| Trigger OSD | Select how the OSD menu is opened with the remote control. |
| Camera Selector | Assign a number to your camera that corresponds to the Camera Select buttons on the remote control. When All Channel is selected, no selection is required on the remote control to operate your camera |
| System Log Download | Click to download system log. |
| Change Cable | Select a lens cable to be installed. When prompted to reboot, disconnect from power and change the lens cable. |


Theme Mode

Select a system mode based on your usage.

| Mode | Video Quality | Behavior |
|------|---|--|
| AVer | Standard | Default |
| Zoom | Zoom certified, suitable for Zoom Rooms | <ul style="list-style-type: none">• IR cut filter mode: Auto• IR cut filter sensitivity: High• Resetting to factory default while in Zoom Mode will load Zoom Mode settings and override all user settings.• Sleep Mode: The camera rotates towards the I/O ports and the LED indicator shows a purple light. |

Room Management

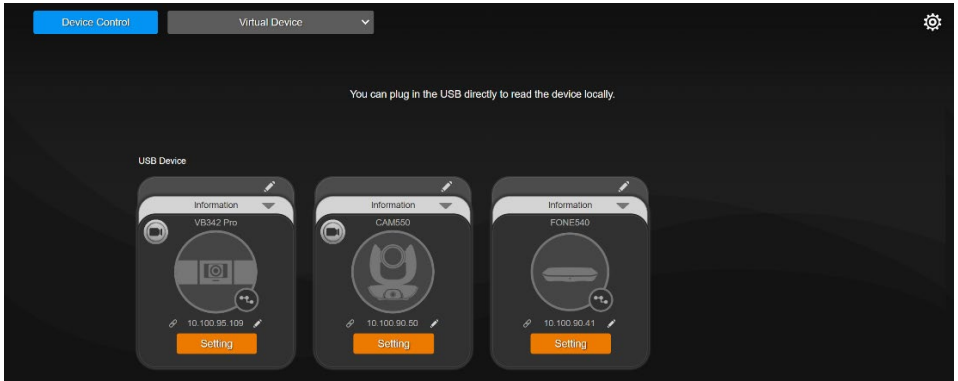Configure Settings on Room Management

1. Download and install Room Management to your computer from AVer Download Center (<https://www.aver.com/download-center>) and launch the software.
2. Connect your device to a computer with the included USB cable. Your device should appear under the **Device Control** page.
3. Click **Setting** to control System settings.

Specifications

| Camera | |
|----------------------------|--|
| Image Sensor | 1/2.8" Exmor CMOS |
| Effective Picture Elements | 8 Megapixels |
| Output Resolutions | 4K/30 (IP/UVC only), 1080p/60, 1080p/59.94, 1080p/50, 1080p/30, 1080p/29.97, 1080p/25, 720p/60, 720p/59.94, 720p/50, 720p/30 |
| Minimum Illumination | 0.7 lux (IRE50, F1.6, 30fps) |
| S/N Ratio | ≥ 50dB |
| Gain | Auto, Manual |
| Shutter Speed | 1/1 to 1/10,000 sec |
| Exposure Control | Auto, Manual, Priority AE (Shutter, IRIS), BLC, WDR |
| White Balance | Auto, Manual |
| Optical Zoom | 30X |
| Viewing Angles | DFOV: 75.5° (Wide) to 3.1° (Tele) HFOV: 68.0° (Wide) to 2.7° (Tele) VFOV: 41.7° (Wide) to 1.5° (Tele) |
| Focal Length | f = 4.3 mm (Wide) to 129 mm (Tele) |
| Aperture (Iris) | F = 1.6 (Wide) to 4.7 (Tele) |
| Minimum Working Distance | Wide: 0.01 m, Tele: 1.2 m |
| Pan / Tilt Angles | Pan: ±170°, Tilt: +90° / -35° |
| Pan / Tilt Speed (Manual) | Pan: 0.1° to 100°/sec, Tilt: 0.1° to 100°/sec |
| Preset Speed | Pan: 200°/sec, Tilt: 200°/sec |
| Preset Position | 10 (IR), 256 (RS-232 / IP) |
| Camera Control - Protocols | VISCA (RS-232 / IP), PELCO-D & PELCO-P (RS-232), CGI (IP) |
| Image Processing | Noise Reduction (2D / 3D), Flip, Mirror |
| Power Frequencies | 50Hz / 59.94Hz / 60Hz |
| Speaker | |
| Embedded Speaker | Yes |
| SPL | 77 dB SPL (1W; 1m) |
| Maximum Output | 90 dB SPL at 0.5 meter |
| Microphone | |
| Built-in Microphone | 2 (Omnidirectional) |
| AI Noise Reduction | Yes |

| | |
|---------------------------------------|--|
| Pickup Range | 5 meters |
| Sensitivity | -33 dBFS |
| Frequency Response | 20 Hz to 20,000 Hz |
| Maximum SPL | 133 dB SPL at 0.5 meter |
| Interface | |
| Video Outputs | HDMI, IP, USB |
| Audio Outputs | HDMI, IP, USB |
| microSD Card Slot | Max. 32GB / SDXC |
| General | |
| Power Requirement | 100-240V AC~ to 12V DC === 3.34A * The included Power Adapter (GSM40A12) is manufactured by MEAN WELL Enterprises Co., Ltd. with Class I classification. ** The power supply is specified as a part of ME EQUIPMENT. |
| Power Input | 12V DC === 3A |
| Power Consumption | 36W |
| Dimension (W x H x D) | 168 x 205 x 126 mm |
| Net Weight | 1.95kg |
| Application | Indoor |
| Tally Lamp | N/A |
| Security | Kensington Slot |
| Remote Control | Infrared |
| Operating Conditions | Temperature: 0°C to +35°C; Humidity: 20% to 80% Atmospheric Pressure: 700 ~ 1060hpa |
| Storage and Transportation Conditions | Temperature: -20°C to +60°C; Humidity: 20% to 95% Atmospheric Pressure: 500 ~ 1060hpa |
| IP Streaming | |
| Resolution | 4K, 30fps |
| Network Video Compress Formats | H.264, H.265 |
| Maximum Frame Rate | 4K, 30fps |
| Network interfaces | 10 / 100 / 1000 Base-T |
| Multi-Stream Capability | 2 (RTSP / Web Page), 4K 30fps (max.) |
| Network Protocols | IPv4, TCP, UDP, ARP, ICMP, IGMP, HTTP, DHCP, RTP / RTCP, RTSP, RTMP, VISCA over IP |
| Recording | |
| Cyclic Threshold Rule | Video stream: 720p, 1080p, 4K 32Mbps: 1.3GB |

| | |
|----------------------------------|--------------------------------------|
| | 16Mbps: 700MB Others: 500MB |
| USB | |
| Connector | USB 3.0 |
| Video Format | MJPEG, YUY2 |
| Audio Format | PCM |
| USB Video Class (UVC) | Video: UVC1.1 Audio: UAC1.0 |
| Software Tools | |
| IP Search and Configuration Tool | AVer ViewCare , AVer Room Management |

Specifications are subject to change without prior notice.

Troubleshoot

What does medical grade mean?

It means the MD330U Series PTZ Camera is certified under IEC 60601-1-2.

This certification guarantees that the MD330 Series PTZ Camera effectively limits the generation, propagation, and reception of electromagnetic energy. By doing so, it minimizes the risks of undesirable effects like electromagnetic interference (EMI) or physical damage to other operational equipment.

Can I install the device upside down?

Yes, make sure to shorten the length of the lens cable by coiling and tying it with a cable tie. Configure Mirror and Flip functions as needed.

How long is the lens cable?

The lens cable is 180cm.

Can I use an extension USB cable to lengthen the lens cable?

No, doing so may damage the device. Please contact AVer for optional accessories.

Can I disinfect the device by spraying alcohol on it?

No, please refer to <[How to Clean and Disinfect](#)>.

How to open the on-screen display (OSD) menu?

1. Make sure the HDMI cable is connected to your device and display.
2. Press and hold the **Menu**  button on the remote control for 3 seconds to open the OSD menu.
3. A solid purple light on the LED indicator will indicate that the OSD menu is turned on.

The device suddenly fails to receive my voice and the LED indicator is flashing orange.

- A flashing orange light on the LED indicator indicates that the microphone is muted.
- Upgrade to firmware 1.1.1030.0 or later to avoid video conferencing software, such as Microsoft Teams or Google Meet, from muting the device when it deems the received volume too high.

Firmware upgrade and rollback.

- To upgrade:
Firmware 1.1.1017.0 or later can upgrade to the latest version.
Firmware 1.1.1016.0 or before must first upgrade to firmware 1.1.1017.0 before upgrading to the latest version.
- To roll back:
The earliest version you can roll back to is firmware 1.1.1017.0.

The image is distorted or blurry.

Do any of the following:

- Autofocus once using the remote control: Press **Enter**.
- Adjust contrast to reduce distortion using the remote control: Press and hold **Menu**  for 3 seconds > **Camera** > **Contrast**.
- Reset the device to factory default settings using the remote control: Press and hold **Menu**  for 3 seconds > **System** > **Factory Default** > **On**.

The speaker audio is staticky or has unwanted noise.

Do any of the following:

- Keep the device at least one meter away from other electrical equipment.
- Adjust the speaker volume and enable noise suppression: On the device's web interface, go to **Video & Audio** > **Audio Setting**.
- Reset the device to factory default settings using the remote control: Press and hold **Menu**  for 3 seconds > **System** > **Factory Default** > **On**.

If the issue persists, contact your authorized distributor or place of purchase for assistance.

No snapshots on the microSD card.

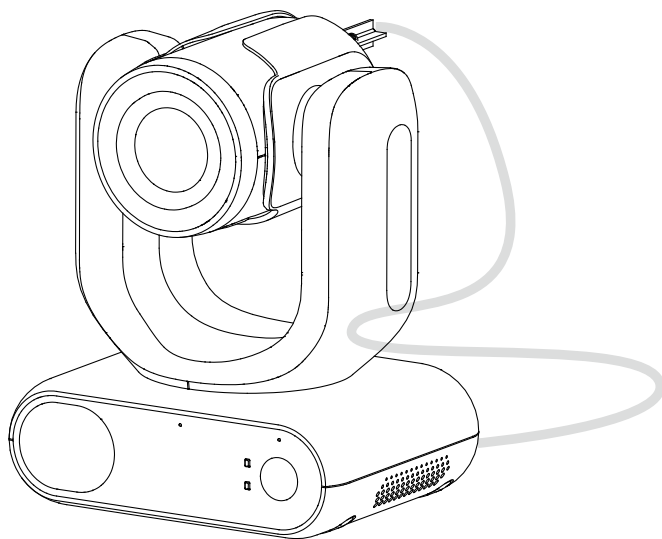
- Make sure a microSD card has been inserted in the slot.
- Press and hold the snapshot button for 1 second to take a snapshot. Press the snapshot button once to autofocus.

The device sometimes disappears from the computer after it's been on for a long time.

In rare cases, the device may disconnect from the computer after extended use. It will automatically recover within about 40 seconds.

During this time, the device may temporarily disappear from your software. Once it reappears, you may need to manually reselect the device—but no restart is needed.

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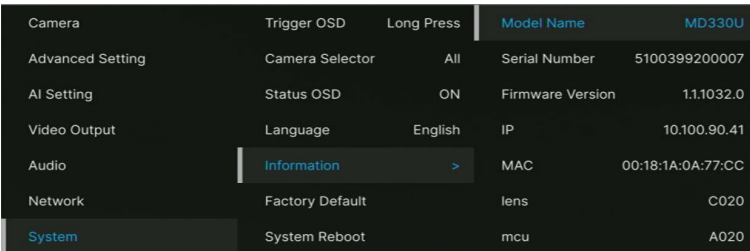
The screenshot shows the 'System > Information' menu. The 'System' option is highlighted in blue. The 'Information' option is selected, showing a list of system details.

| | | | | |
|------------------|-----------------|------------|------------------|-------------------|
| Camera | Trigger OSD | Long Press | Model Name | MD330U |
| Advanced Setting | Camera Selector | All | Serial Number | 5100399200007 |
| AI Setting | Status OSD | ON | Firmware Version | 1.11032.0 |
| Video Output | Language | English | IP | 10.100.90.41 |
| Audio | Information | > | MAC | 00:18:1A:0A:77:CC |
| Network | Factory Default | | lens | C020 |
| System | System Reboot | | mcu | A020 |

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| 第一層 | 第二層 | 第三層 | 第四層 |
|---------------------------|---------------|-------------------|--------------------------|
| Camera | Exposure Mode | Full Auto | Exposure Value |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | | WDR |
| | | Shutter Priority | Exposure Value |
| | | | Shutter Speed |
| | | | Gain Limit Level |
| | | | BLC |
| | | Iris Priority | Exposure Value |
| | | | Iris Level |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | Manual | Iris Level |
| | | | Shutter Speed |
| | Gain Level | | |
| | White Balance | Auto | |
| | | ATW | |
| | | Indoor | |
| | | Outdoor | |
| | | One push | |
| | | Manual | R gain |
| | | B gain | |
| | Pan Tilt Zoom | Preset Speed | 5, 25, 50, 100, 150, 200 |
| | | Preset Accuracy | OFF / ON |
| | | Sensor Zoom | OFF / ON |
| | | Sensor Zoom Limit | x2 |
| | | Pan Speed | 1~24 |
| | | Tilt Speed | 1~24 |
| | | Zoom Speed | Low / High |
| P/T Spd. Relative Z Ratio | | OFF / ON | |

| | | | |
|------------------|-----------------------|---------------------------|-----------------------------|
| | | Pan L/R Dir. Switch | OFF / ON |
| | | Focus Mode | Manual / Auto |
| | Noise filter | Off / Low / Middle / High | |
| | Saturation | 0 1 2 3 4 5 6 7 8 9 10 | |
| | Contrast | 0 1 2 3 4 | |
| | Sharpness | 0 1 2 3 | |
| | Mirror | OFF / ON | |
| | Flip | OFF / ON | |
| Advanced Setting | Control | Type | RS232 |
| | | Protocol | VISCA / PELCO D/PELCO P |
| | | Camera Address | 1 2 3 4 5 6 7 |
| | | Baud Rate | 2400 / 4800 / 9600 / 115200 |
| | IR Cut Filter | Auto / Day / Night | |
| | IR Cut Sensitivity | Low / Middle / High | |
| | Performance Mode | OFF / ON | |
| | AI Mosaic | OFF / NET / HDMI | |
| | Mosaic Type | Body / Full Face | |
| AI Setting | Eyes Tracking | Eyes Tracking | OFF / ON |
| | | Tracking Site | Save |
| | | Tracking Type | Full Face/Eyes |
| | | Tracking Range | Close / Medium / Wide |
| | | Timeout to Site | 3/5/7/10 sec |
| | Detection | Detection | OFF / ON |
| | | Detection Type | Body / Full Face |
| | Fall/Egress Detection | Fall/Egress Detection | OFF / ON |
| Detection Type | | All Type / Fall / Egress | |
| Video Output | Frequency | 60 | |
| | | 59.94 | |
| | | 50 | |
| | Resolution | 1080p60 | |
| | | 1080p59.94 | |
| | | 1080p30 | |
| | | 1080p29.97 | |
| | | 720p60 | |
| | 720p59.94 | | |

| | | | |
|------------------|-----------------------------------|---|-----------------------------------|
| | | 1080p50 | |
| | | 1080p25 | |
| | | 720p50 | |
| | | 720p30 | |
| Audio | Audio Input | Noise Suppression | OFF/ NR DSP/ NR/ NR+BF/ NR Strong |
| | | Audio Beam Forming | 30,50,70,90,110,130 |
| | | Audio In Volume | Mute / 1~10 |
| | Audio Output | Output Type | Speaker/HDMI/Both |
| | | Output Volume | Mute / 1~10 |
| Network | DHCP | OFF / ON | |
| | Static IP | IP Address | 192.168.1.168 |
| | | Gateway | 192.168.1.254 |
| | | Mask | 255.255.255.0 |
| | | DNS | 8.8.8.8 |
| System | Trigger OSD | Click Menu to open, Press Menu 3 sec | |
| | Camera Selector | 1,2,3,4, All channel, Disable Remote | |
| | Status OSD | OFF / ON | |
| | Language | English / 中文 / 日本語 | |
| | Change cable | Standard cable / Long cable | |
| | Information | Model Name | MD330U(l) |
| | | Serial number | xxxxxxxxxxxxxx |
| | | Firmware Version | 0.0.0000.00 |
| | | IP | 192.168.1.168 |
| | | MAC | 00:18:1a:04:9e:81 |
| | | lens | xxxx |
| | | mcu | xxxx |
| | | SD Total (MB) | |
| | | SD Free (MB) | |
| Recording Status | Stopped / xx:xx:xx (record timer) | | |
| Factory Default | OFF / ON | | |
| System Reboot | OFF / ON | | |
| Theme Mode | AVer/Zoom | | |

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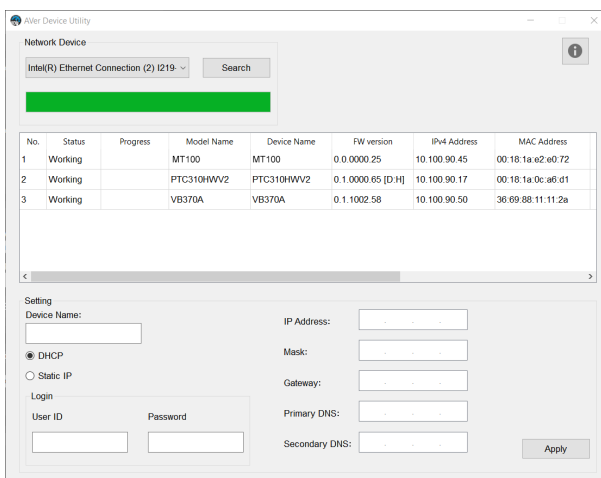
AVer Device Utility

The screenshot shows the AVer Device Utility application window. At the top, there is a search bar with a dropdown menu set to 'Intel(R) Ethernet Connection (2) I219' and a 'Search' button. Below the search bar is a green progress bar. A table lists three network devices with their status, progress, model names, device names, firmware versions, IP addresses, and MAC addresses. At the bottom, there are configuration settings for the selected device, including fields for Device Name, IP Address, Mask, Gateway, Primary DNS, and Secondary DNS. The DHCP option is selected, and there are fields for User ID and Password. An 'Apply' button is located at the bottom right.

| No. | Status | Progress | Model Name | Device Name | FW version | IPv4 Address | MAC Address |
|-----|---------|----------|------------|-------------|-------------------|--------------|-------------------|
| 1 | Working | | MT100 | MT100 | 0.0.0000.25 | 10.100.90.45 | 00:18:1a:e2:e0:72 |
| 2 | Working | | PTC310HWV2 | PTC310HWV2 | 0.1.0000.65 [D.H] | 10.100.90.17 | 00:18:1a:0c:a6:d1 |
| 3 | Working | | VB370A | VB370A | 0.1.1002.58 | 10.100.90.50 | 36:69:88:11:11:2a |

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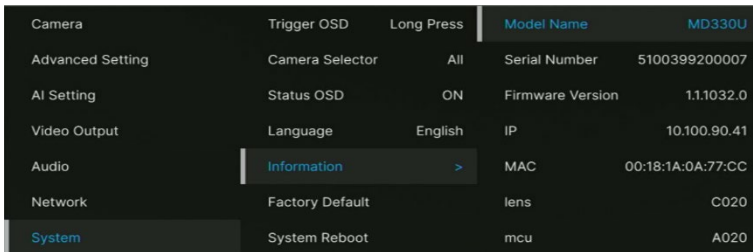
OSD Menüstruktur

| 1. Ebene | 2. Ebene | 3. Ebene | 4. Ebene |
|---------------------------|---------------|-------------------|--------------------------|
| Camera | Exposure Mode | Full Auto | Exposure Value |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | | WDR |
| | | Shutter Priority | Exposure Value |
| | | | Shutter Speed |
| | | | Gain Limit Level |
| | | | BLC |
| | | Iris Priority | Exposure Value |
| | | | Iris Level |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | Manual | Iris Level |
| | | | Shutter Speed |
| | Gain Level | | |
| | White Balance | Auto | |
| | | ATW | |
| | | Indoor | |
| | | Outdoor | |
| | | One push | |
| | | Manual | R gain |
| | | B gain | |
| | Pan Tilt Zoom | Preset Speed | 5, 25, 50, 100, 150, 200 |
| | | Preset Accuracy | OFF / ON |
| | | Sensor Zoom | OFF / ON |
| | | Sensor Zoom Limit | x2 |
| Pan Speed | | 1~24 | |
| Tilt Speed | | 1~24 | |
| Zoom Speed | | Low / High | |
| P/T Spd. Relative Z Ratio | | OFF / ON | |

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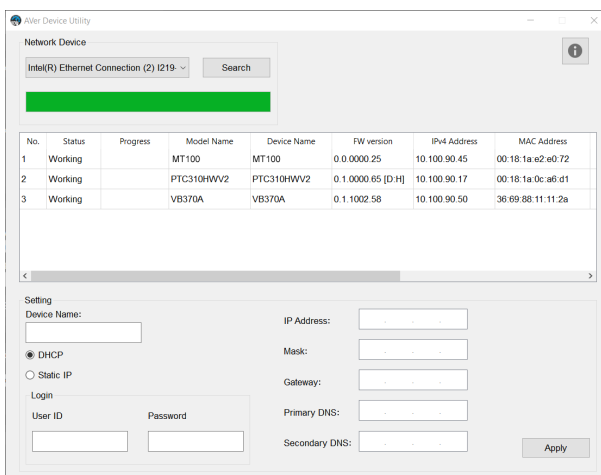
AVer Device Utility

The screenshot shows the AVer Device Utility application window. At the top, there is a 'Network Device' section with a dropdown menu set to 'Intel(R) Ethernet Connection (2) I219' and a 'Search' button. Below this is a green progress bar. A table lists three network devices with their status, progress, model names, device names, firmware versions, IPv4 addresses, and MAC addresses. At the bottom, there is a 'Setting' section with fields for 'Device Name', 'IP Address', 'Mask', 'Gateway', 'Primary DNS', and 'Secondary DNS'. There are radio buttons for 'DHCP' (selected) and 'Static IP', and a 'Login' section with 'User ID' and 'Password' fields. An 'Apply' button is located at the bottom right.

| No. | Status | Progress | Model Name | Device Name | FW version | IPv4 Address | MAC Address |
|-----|---------|----------|------------|-------------|-------------------|--------------|-------------------|
| 1 | Working | | MT100 | MT100 | 0.0.0000.25 | 10.100.90.45 | 00:18:1a:e2:e0:72 |
| 2 | Working | | PTC310HWV2 | PTC310HWV2 | 0.1.0000.85 [D-H] | 10.100.90.17 | 00:18:1a:0c:a6:d1 |
| 3 | Working | | VB370A | VB370A | 0.1.1002.58 | 10.100.90.50 | 36:69:88:11:11:2a |

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Albero del menu OSD

| 1° Livello | 2° Livello | 3° Livello | 4° Livello |
|---------------------------|---------------|-------------------|--------------------------|
| Camera | Exposure Mode | Full Auto | Exposure Value |
| | | | Gain Limit Level |
| | | | Slow Shutter |
| | | | BLC |
| | | | WDR |
| | | Shutter Priority | Exposure Value |
| | | | Shutter Speed |
| | | | Gain Limit Level |
| | | | BLC |
| | | Iris Priority | Exposure Value |
| | | | Iris Level |
| | | | Gain Limit Level |
| | Slow Shutter | | |
| | BLC | | |
| | Manual | Iris Level | |
| | | Shutter Speed | |
| | | Gain Level | |
| | White Balance | Auto | |
| | | ATW | |
| | | Indoor | |
| | | Outdoor | |
| | | One push | |
| | | Manual | R gain |
| | | B gain | |
| | Pan Tilt Zoom | Preset Speed | 5, 25, 50, 100, 150, 200 |
| | | Preset Accuracy | OFF / ON |
| | | Sensor Zoom | OFF / ON |
| | | Sensor Zoom Limit | x2 |
| Pan Speed | | 1~24 | |
| Tilt Speed | | 1~24 | |
| Zoom Speed | | Low / High | |
| P/T Spd. Relative Z Ratio | | OFF / ON | |

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